


Quickstart Guide

A&D Weight Scale



Step ONE:


Insert batteries. Open the lid on the back of the scale and insert the batteries.

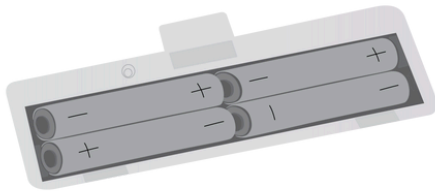

Note: Once the batteries are inserted, it may take several minutes for the scale to connect with the cellular network to set the clock. The length of time will depend on the network strength in your area. Do not power off.

Step TWO:

Place the scale on a hard, even surface. Ensure scale is located in an area with good cellular signal. If scale is not connecting, move closer to a window or an area with strong cellular network connection.

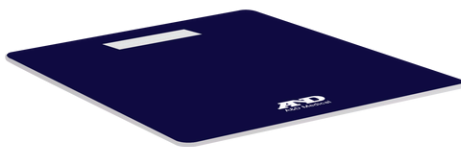

Network is Connected



Network is NOT Connected:

- Move closer to a window or an area with a stronger network connection. Wait and take another measurement.
- If scale does not connect, contact support.



For further assistance, please contact your care team.

Read the device manufacturer instructions for operation and safety before use.

Step THREE:

Take a reading. Step on the scale and remain still. Step off the scale when “StEP” and “oFF” are alternately displayed with three beeps.

**Step FOUR:**

Measurement is sent to your healthcare provider. A cloud with a check mark will appear on the screen when a measurement is sent.



For further assistance, please contact your care team.

Read the device manufacturer instructions for operation and safety before use.