


Quickstart Guide

Tenovi Hub



Step ONE

Using the provided power cord, plug the pronged end into a power outlet, and the smaller end into the side of the Tenovi Gateway Hub. Wait for the light on the Hub to turn red.

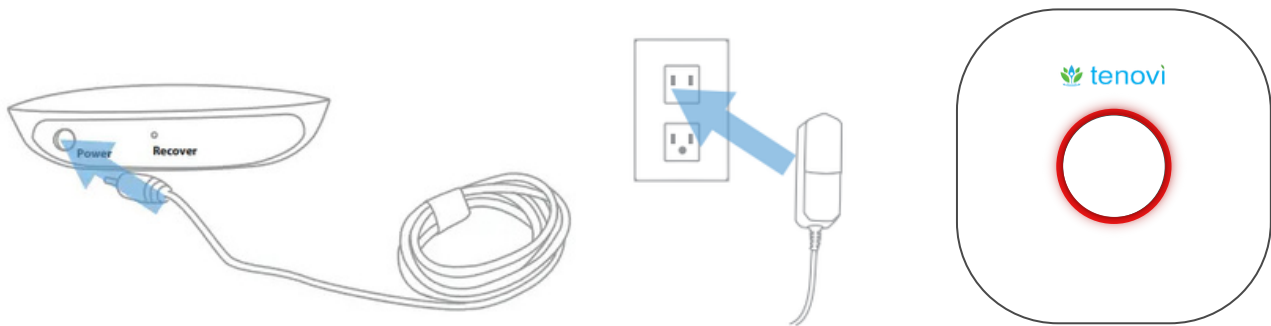

Step TWO

Take a daily device measurement, or as directed by your care team. The Hub will turn YELLOW when a reading is being taken, and turn GREEN when the reading is transmitted successfully.



YELLOW:
a reading is
being taken



GREEN:
the reading is
transmitted
successfully.

For further assistance, please contact your care team.



Read the device manufacturer instructions for operation and safety before use.

COLOR INDICATOR GUIDE



Green

If the light ring is green, this indicates you have taken your readings for the day.



Red

If the light ring is RED, this indicates you need to take measurement readings for the day.




Yellow

If the light ring remains yellow, this means your readings have not been transmitted. Wait 15 minutes. If still yellow, move the hub to another location with better signal.



Flashing Blue

If the Hub flashes blue for more than 30 seconds, unplug the power adapter, wait 5 seconds, and plug the power adapter back into the Hub.

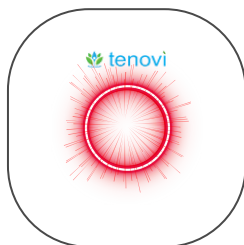

Flashing Red

If the LED ring flashes RED after you take a reading, you have triggered a measurement reading alert. If this happens, take one more reading.

For further assistance, please contact your care team.



Read the device manufacturer instructions for operation and safety before use.